
Wrist / Palm Support with Thumb Splint



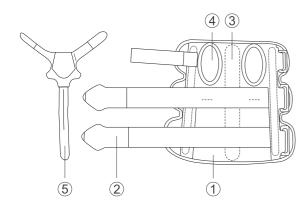


- 1. Breathable fabric for more comfort.
- 2. Hook-side fastener for easy adjustment.
- 3. Shapeable aluminum alloy stays stabilize palm.
- 4. Suitable for both left & right hands.
- 5. Plug-in thumb splint for extra support.

Material Composition

- 60% Nylon
- 15% Aluminum
- 13% Polyester
- 8% PU foam
- 4% PP

Size chart

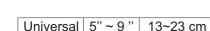



KAO CHEN ENTERPRISE CO., LTD.


NO.68, LANE326, SANGANG RD., LONGJING, TAICHUNG 43444, TAIWAN Tel: 886-4-26308728 Fax: 886-4-26308729 www.softguards.com E-mail : service@softguards.com






Instruction


1. Loosen all straps, insert thumb splint into lateral side of brace. Ensure thumb splint in correct position (for left or right).

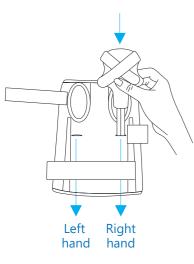


2. Slide palm & thumb into the brace with the splint positioned along the palm & thumb.

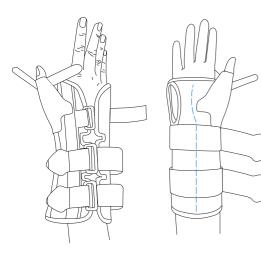

- 3. a. Wrap straps #1 & #2 to proper tightness and then attach hook-side fastener onto the straps.
 - b. Have strap #3 & #4 to wrap tight and attach the hook-side fastener to strap.
 - c. Adjust #5 strap to suitable tightness and then attach to hook-side fastener.
 - d. Double check the overall tightness before cut off the excess part of #5 strap for proper length.

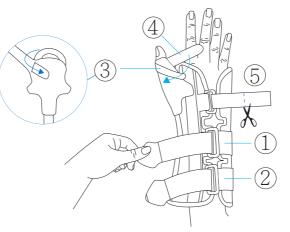

Product views

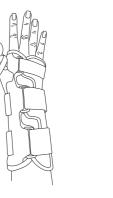



Opisthenar

Palm

Caution

- 1. The product should be used under the supervision of a medical professional.
- 2. Over tightening of strap may compromise blood circulation.
- 3. If discomfort or pain results and persists, please discontinue wearing and consult with medical professional.
- 4. Make sure the fasteners are well fastened while wearing the brace. If the fasteners can not be fastened well, please stop using the brace to prevent injuries.
- 5. If product is in poor condition, please do not use it.

Washing instructions

- 1. Separate the aluminum stay from the brace before washing.
- 2. Hand wash with mild detergent.
- 3. Rinse thoroughly and air dry.
- 4. Avoid direct exposure to sunlight.



Opisthenar

Palm